

# Bone tissue augmentation

MinerOss™ A  
An allograft that offers an  
alternative treatment option  
to autologous graft.

Valid from May 2021



# Safety aspects

## Serological testing

Virus	Test	Specification
Hepatitis B virus (HBV)	HBsAg, HBcAb*, NAT	negative
Hepatitis C virus (HCV)	Ab, NAT	negative
Human immunodeficiency virus	Ab, NAT	negative
Human T-Lymphotropic virus (HTLV 1/2)	Ab, NAT	negative



Bacteria	Test	Specification
Treponema pallidum (Lues)	CMIA, TP Ab	negative

Only tissue of donors who have been tested negative enters the Allotec® process.

\* negative for active infection (in case of positive HBcAb C+TBA performs further investigations)

## The Allotec® process

### Independent validation of the critical viral inactivation steps of the process

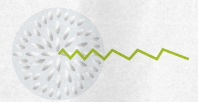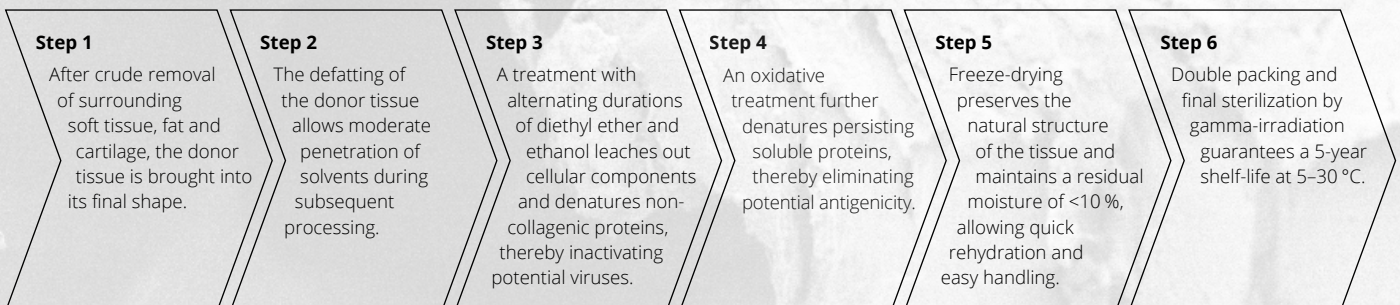



# Manufacturing process of MinerOss™ A

## High safety standards

After thorough donor anamnesis, maximum safety is assured via a series of strict serological testing combined with C+TBA's Allotec® purification procedure and radiological sterilization.

First, serology and nucleic acid tests (NAT) are performed to identify infections before antibodies are detected in the blood. Potential viruses are inactivated, and bacteria destroyed during Allotec® purification procedure where non-collagenic proteins are denatured.



An oxidative treatment is then used to denature persisting soluble proteins and eliminate potential antigenicity. Finally, the tissue undergoes dehydration via a lyophilization technique that facilitates the sublimation of frozen water from solid phase to gas phase. This stage preserves the material's structural integrity.

The lyophilization process is an accepted and well document procedure that preserves structural properties while improving graft incorporation. [7, 8]


The microscopic pores within the material ensure rapid rehydration of the tissue. The final sterilization by gamma irradiation guarantees a sterility assurance level (SAL) of  $10^{-6}$  while ensuring structural and functional integrity of the product and its packaging.

## The success factors of the manufacturing process

### The Allotec® process

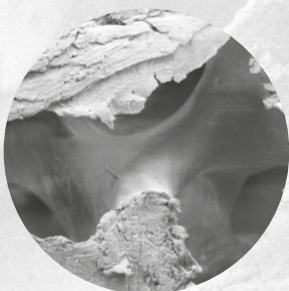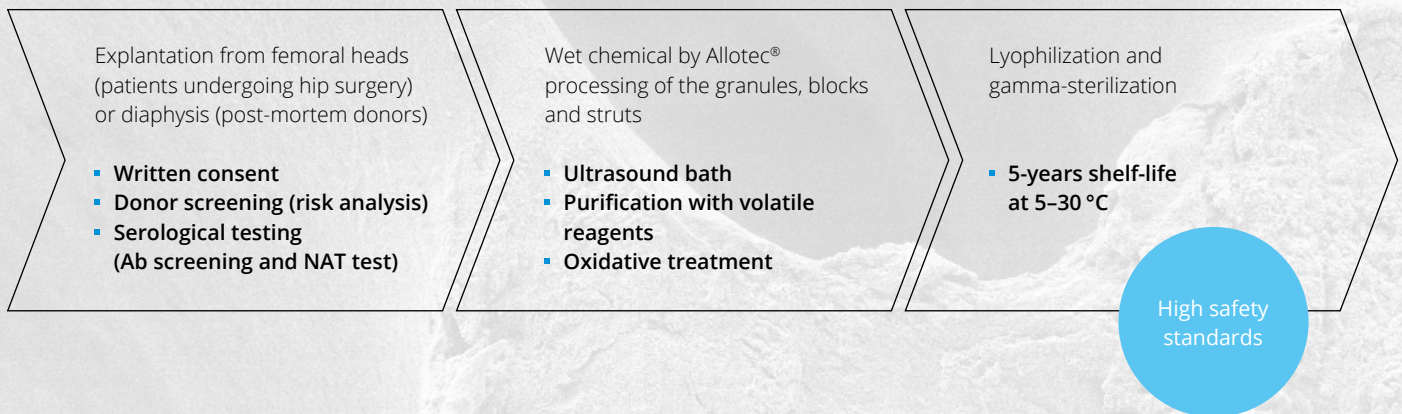

SEM picture of MinerOss™ A at 100-fold magnification showing macroporous structure.

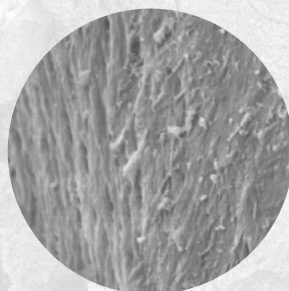

SEM picture of MinerOss™ A at 500-fold magnification showing microporous structure.



# MinerOss™ A human bone substitute

## Fast graft incorporation and complete remodeling potential [1–5]

The scientific evidence shows that allografts are the second best option to patient's own bone compared to other bone substitutes. [1]

MinerOss™ A is a processed allogenic bone tissue that offers proven, reliable, and predictable surgical outcomes comparable to autologous bone harvested from intra-oral donor sites.

MinerOss™ A is an allograft predominantly derived from human donor femoral heads post hip replacement surgery. The donor tissue is thoroughly inspected and undergoes a strict serological screening protocol.

Thanks to its natural bone composition consisting of mineralised human collagen, MinerOss™ A shows a high biologic regeneration capability in combination with natural remodelling. Therefore MinerOss™ A is an excellent alternative to autologous bone, meaning that there is no need for an intraoral surgical donor site, which reduces morbidity for the patient.

MinerOss™ A undergoes high safety standards in its production process and guarantees a 5-year shelf life at room temperature (5–30 °C).



**Post-mortem donor – diaphysis:**  
MinerOss™ A Cortico-cancellous Block and  
MinerOss™ A Cortical Strut

**Living donors – femoral heads:**  
MinerOss™ A Cancellous Granulate,  
MinerOss™ A Cortico-cancellous Granulate and  
MinerOss™ A Cancellous Block

# Product characteristics of MinerOss™ A


## Proprietary tissue processing maintains tissue integrity

- Bone from human donors  
(living donors: femoral heads, post-mortem donors: diaphysis)
- Natural bone composition – mineralized human collagen
- High biologic regeneration capability and natural remodeling [4]
- High safety standards
- 5 years shelf-life at room temperature (5–30 °C)




## Histological structure of MinerOss™ A

### Histology of biopsy harvested 7 months after block augmentation [3, 5, 6]

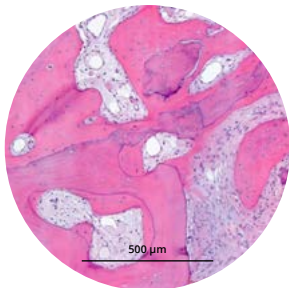

The allogenetic bone can be recognized by the empty cavities of the osteocytes and is embedded in newly formed bone matrix.

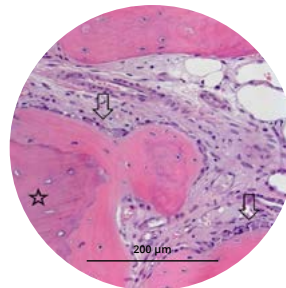

Allogenetic bone (asterisk) embedded in newly formed bone. New bonematrix is lined by osteoblast (arrows) showing ongoing bone formation.

# Ideal for following indications

## Regeneration and augmentation

Large bone defects where one bone wall is maintained can be restored to replace missing or inadequate bone tissue, or for filling or stabilizing bone defects by using MinerOss™ A in combination with collagen membrane.

- Regeneration of periodontal osseous defects
- Regeneration after cyst and root tip resections
- Regeneration of extraction sockets
- Regeneration of gaps between alveolar wall and dental implants
- Sinus augmentation
- Regeneration of gaps around block grafts
- Horizontal augmentation of alveolar ridges
- Three dimensional (horizontal and/or vertical) augmentation of alveolar ridges



### Regeneration of extraction sockets

Filling the socket with MinerOss™ A in combination with collagen membrane regenerates bone and so retains the volume and shape of the bone over time.



### Regeneration of periodontal osseous defects

A tooth with a good prognosis can be retained by regenerating lost bone, with the support of biomaterials such as MinerOss™ A in combination with collagen membrane.



### Regeneration of gaps between alveolar wall and dental implants

MinerOss™ A is applied to the defect to support bone regeneration.

# The alternative bone-tissue graft

High patient acceptance – shorter treatment times

- **Shorter surgery time**

The ready-to-use allograft shortens surgery time by eliminating the need for a donor site.

- **Lower patient morbidity and less pain for the patient**

Avoiding a donor site eliminates the postoperative pain associated with a second procedure.

- **Unlimited availability**

It is predominantly derived from human donor femoral heads post hip replacement surgery.

- **No clinical difference in final incorporation compared to autologous bone [3, 5, 6]**

The scientific evidence shows that allografts are the second best option to patient's own bone compared to other bone substitutes [1]



# Ordering information

## MinerOss™ A



### MinerOss™ A Cancellous Granulate



Art. No.	Volume	Particle size
<b>BM1007.1005</b>	0.5 cm <sup>3</sup>	250–1000 µm
<b>BM1007.1010</b>	1.0 cm <sup>3</sup>	250–1000 µm
<b>BM1007.1020</b>	2.0 cm <sup>3</sup>	250–1000 µm
<b>BM1007.1040</b>	4.0 cm <sup>3</sup>	250–1000 µm

### MinerOss™ A Cortico-cancellous Granulate

Art. No.	Volume	Particle size
<b>BM1008.1005</b>	0.5 cm <sup>3</sup>	250–1000 µm
<b>BM1008.1010</b>	1.0 cm <sup>3</sup>	250–1000 µm
<b>BM1008.1020</b>	2.0 cm <sup>3</sup>	250–1000 µm
<b>BM1008.1040</b>	4.0 cm <sup>3</sup>	250–1000 µm

### MinerOss™ A Cancellous Block

Art. No.	Product size
<b>BM1010.1010</b>	10x10x10 mm
<b>BM1010.1020</b>	10x10x20 mm

### MinerOss™ A Cortico-cancellous Block

Art. No.	Product size
<b>BM1009.1010</b>	10x10x10 mm
<b>BM1009.1020</b>	10x10x20 mm

### MinerOss™ A Cortical Strut

Art. No.	Product size
<b>BM1010.1000</b>	25x10x1 mm

25 x 10 mm (Cortical)

### Responsible tissue bank

Cells+Tissuebank Austria gGmbH | Magnesitstr. 1 | 3500 Krems an der Donau | Austria  
Phone +43 2732 76954 0 | Fax +43 2732 76954 40 | [vigilanz@ctba.at](mailto:vigilanz@ctba.at)

## References

- [1] Schmitt et al. Clin Oral Implants Res. 2013, 24, 576.
- [2] Solakoglu et al. Clin Implant Dent Relat Res. 2019, 21, 1002-1016.
- [3] Kloss et al. Clin Case Rep. 2020, 8, 5.
- [4] Wen et al. J. Periodont. 2019, 1, 734.
- [5] Kloss et al. Clin Oral Implants Res. 2018, 29, 1163
- [6] Tunkel et al. Clinical Case Reports 2020, 00:1-13.
- [7] Trajkovski et al. Materials 2018, 11(2).
- [8] Barbeck et al. Materials 2019, 12, 3234.

### Headquarters

CAMLOG Biotechnologies GmbH | Margarethenstr. 38 | 4053 Basel | Switzerland  
Phone +41 61 565 41 00 | Fax +41 61 565 41 01 | [info@camlog.com](mailto:info@camlog.com) | [www.camlog.com](http://www.camlog.com)

MinerOss™ A is manufactured by C+TBA. Allotec® is a registered trademark of C+TBA. MinerOss™ is a trademark of BioHorizons. BioHorizons® is a registered trademark of BioHorizons. They may, however, not be registered in all markets. As applicable, BioHorizons products are cleared for sale in the European Union under the EU Medical Device Directive 93/42/EEC and the human tissues and cells Directive 2007/47/EC. We are registered to ISO 13485:2016, the international quality management system standard for medical devices, which supports and maintains our product licenses with Health Canada and in other markets around the globe. Original language is English. ©BioHorizons. All rights reserved. Not all products shown or described in this literature are available in all countries.

